

POCT for Workplace Drug Testing

Point of Collection Testing (POCT) devices to detect drugs of abuse in human urine.



Products Available

For Workplace Drug Testing, an integrated Split Key cup is available with two different configurations.

- 1) Name: **Split Specimen 6 Panel Cup w/Adulteration**
 Drugs: (AMP/THC/COC/mAMP/MOP/PCP)
 PAML Part #: 8355
- 2) Name: **Split Specimen 10 Panel Cup w/Adulteration**
 Drugs: (mAMP/COC/THC/AMP/OPI/OXY/PCP/BZO/BAR MTD)
 PAML Part #: 8354

Drug Menu

Drug Name	Abbreviation	Cutoff Levels
Amphetamine	AMP	1000 ng/mL
Barbiturates	BAR	300 ng/mL
Benzodiazepines	BZO	300 ng/mL
Buprenorphine	BUP	10 ng/mL
Cocaine	COC	300 ng/mL
Marijuana	THC	50 ng/mL
Methadone	MTD	300 ng/mL
Methamphetamine	mAMP	1000 ng/mL
Opiates	OPI	2000ng/mL
Opiates	MOP	300 ng/mL
Oxycodone	OXY	100 ng/mL
Phencyclidine	PCP	25 ng/mL

Features

- ▶ Integrated collection cup and test cassette allows for easy urine collection and prevents tampering.
- ▶ Each cup has a temperature strip affixed to its side to enable users to verify the validity of the specimen as the urine is provided.
- ▶ Test results are kept confidential until the authorized individual is ready to view them.
- ▶ Collectors may record pertinent information regarding the specimen directly on the collection cup to aid in identification.
- ▶ Custom designed test cup provides for easy handling during the collection process.
- ▶ Split specimen for the testing procedure ensures remaining specimen is unaffected in case it is needed for confirmatory testing.
- ▶ Test devices are self-contained and portable and can be sent directly to the laboratory if confirmation is required.
- ▶ Test controls are built into the device.
- ▶ Negative results are available in as low as 3 minutes, with positive results in 5 minutes.
- ▶ Results appear clear and are easy to read.



Contact Us Today at www.paml.com
 Ph: 877-778-9590
toxsales@paml.com
 110 W. Cliff Ave, Spokane, WA 99204


NOBLE

Split-Specimen Cup®

1 Collect Specimen




2 Insert Key

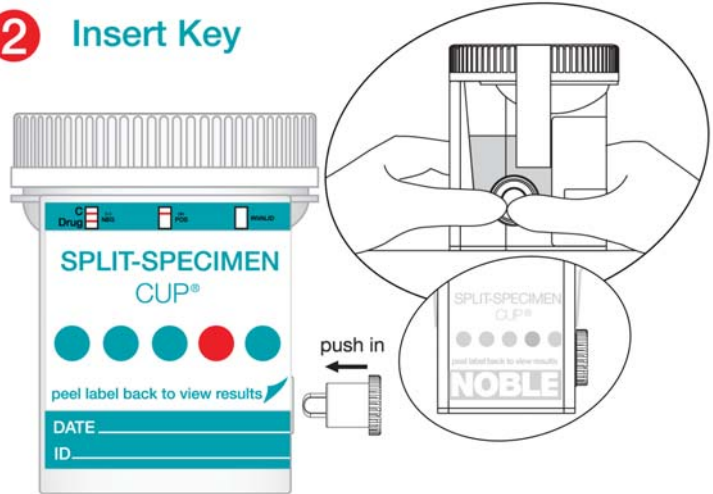

3 Peel Back Label




4 Read the Results

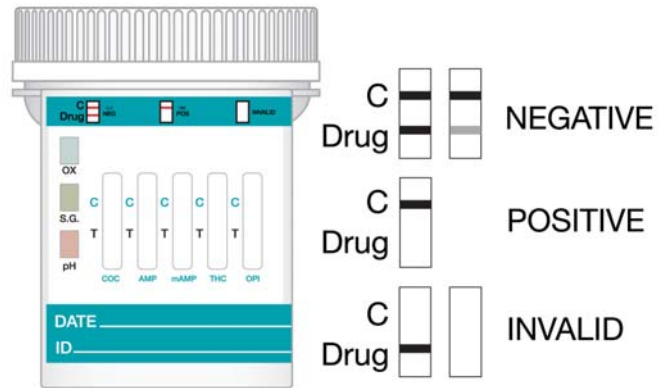

All positive results are presumptive and should be confirmed by an alternate method. (e.g. GC/MS)



Read results at (5) minutes.
Results stable up to (60) minutes.

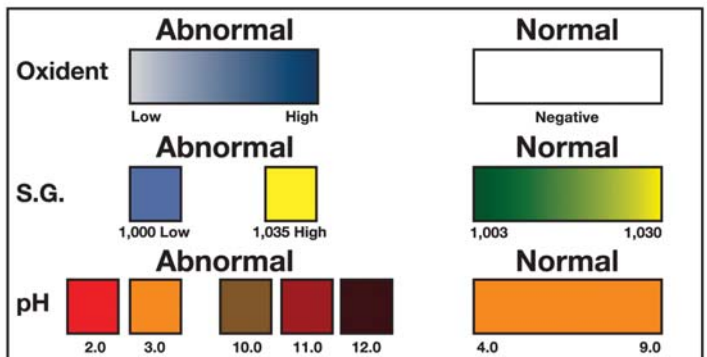

SAMPLE ONLY - Color chart NOT exact.